




Timestrip® Blood Temp 6 (BT6)

Use, Application, & Specifications

Use & Application

1. Select a BT6 indicator and ensure it is not activated: the arming window should be yellow and the breach window should be white. BT6 does not require pre-conditioning.
2. Note: BT6 can be activated at normal room temperature – any temp above 6°C/43°F will do.
3. Firmly squeeze the button located on top of the indicator, between thumb and forefinger.
4. Verify that the arming window changes color immediately from yellow to dark green.
5. Immediately after activation, refrigerate BT6 to temperatures of +3°C/38°F or below for a minimum of 15 minutes.
6. Remove the pre-chilled blood bag from its controlled refrigerator (2°C/36°F – 6°C/43°F).
7. Select a suitable location on the lower third of blood bag where there is always a large volume of blood product. Ensure this area is oil & moisture free.
8. Remove BT6 from refrigeration, peel its liner and firmly adhere the BT6 to the prepared location. Verify full adhesion to the blood bag.

Monitoring has begun!

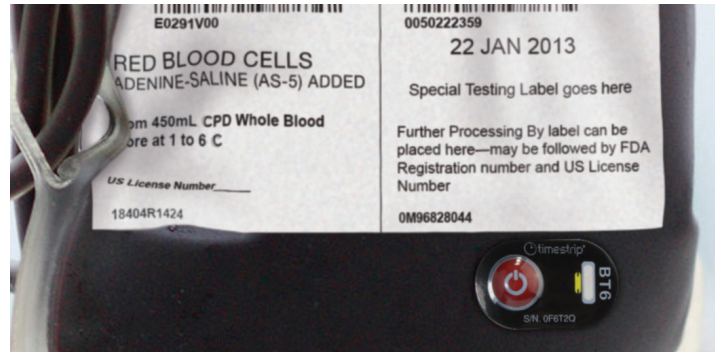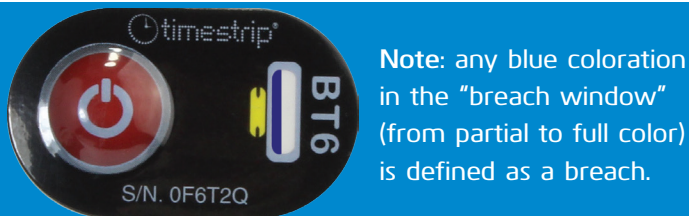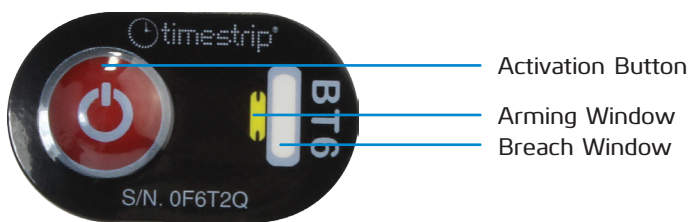

Specifications

A Specification, MSDS and Testing Protocol are available.

Type	Irreversible, single use
Activation Feature	A manual activation button on the top – the activation window turns from yellow to green to confirm it is armed
Indication	A blue color appears in the breach window if a breach of the threshold temperature has occurred. Any blue, whether full or partial, indicates a breach.
Monitored Temperature	Threshold of 6°C/43°F,
Accuracy of Temperature	5–6°C/41–43°F
Size	40mm x 19mm, blister height 0.5mm
Adhesive	Suitable for adherence to a blood bag

Storage

Recommended Storage	Room temperature: 22°C/72°F at 20–80% relative humidity. Cool dark environment
Packaging	Loose in a box. Labeled with expiry and batch number
Shipping Package Requirements	No special requirements as inert until activated
Protection	Splash proof
Traceability	Unique serial number on each indicator
Shelf Life	Pre-activation – 2 years from shipping, post activation 1 year or until breached

This product is not a replacement for the professional judgement of laboratory/ blood bank professionals and/or policies and procedures that are applicable to your institution.

Contact Us Today for More Information & Free Samples!
Call 800.435.4242 or visit www.pdchealthcare.com/bloodtemp10

27770 N. Entertainment Dr., Ste. 200
Valencia, CA 91355
Tel 800.435.4242 or 661.257.0233
Fax 800.321.4409
www.pdchealthcare.com

Distributed by PDC Healthcare

Where positive identification
meets safe, efficient patient care.

