

# V-NOTCH™ SECONDARY LABELS

Reduce specimen relabeling  
by up to 80%



# EASIER, MORE ACCURATE IDENTIFICATION OF BD VACUTAINER® PLUS PLASTIC BLOOD COLLECTION TUBES

Mislabeled specimens or inconsistent barcoding can compromise patient safety, increase costs, and waste valuable staff time and resources. The v-shaped notch of our V-Notch™ Specimen Labels aligns with the color-coded notch and sidebar of BD Vacutainer® Plus tubes to make labeling easier. They provide:

- Consistent & easy labeling guides
- Accurate & reliable bar code scanning
- Operational efficiencies that save time & money
- Increased clinician satisfaction & patient safety



**✘ BEFORE**

Provides correct and consistent label placement on BD Vacutainer® Plus Collection Tubes



**✔ AFTER**

Ensures accurate identification with aligned labels that scan easily and won't peel off unintentionally

Learn more at [pdhealthcare.com/v-notch](https://pdhealthcare.com/v-notch)

# HEAR FROM LAB PERSONNEL



Compatible with most direct thermal printers

“Prior to using V-Notch™, our staff was forced to re-label more than **90%** of our test tubes. Now that we’re using V-Notch™, we’re re-labeling fewer than **10%**. Plus, we’re experiencing significant time and cost savings.”

—Barbara Bush, Laboratory Manager  
[University of Kentucky](#)



Easy Labeling Guides

“After switching to V-Notch™ labels, we increased our number of lab results by 290,000 over 3 years! That’s a **14%** increase in productivity. On top of that, we reduced test tube label costs by **6%**.

V-Notch™ helped us reduce expenses, increase efficiency, and maintain our focus on patient safety and quality care.”

—Dawn D. Moore, Laboratory Director  
[Piedmont Hospital, Atlanta, GA](#)

# 3 EASY STEPS FOR ACCURATE IDENTIFICATION

## STEP 1:



Peel your V-Notch™ Secondary Label from the liner.

## STEP 2:




Visually align the V-Notch™ with the colored notch on the BD Vacutainer® Plus Plastic Tube.

## STEP 3:



Before you apply the V-Notch™ Secondary label, make sure the color-coded notch and the color-coded sidebar on the BD Vacutainer® Plus Plastic Tube are clearly visible. Smoothly wrap the label around the tube.

## Meditech V-Notch™ Labels

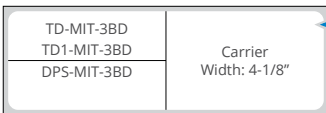


V-Notch



V-Notch



V-Notch

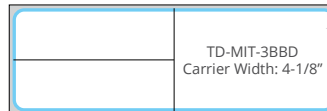


V-Notch

PRODUCT NUMBER	LABEL SIZE	CORE SIZE	QTY.	MATERIAL/ADHESIVE
TD-MIT-3BD	4" X 1-1/4"	3"	3,600/RL/-1RL/BX	Direct Thermal/Permanent
TD1-MIT-3BD		1"	1,000/RL-6RLS/BX	
TD-MIT-3GBD		3"	3,600/RL-1RL/BX	
TD-MIT-3RBD		3"	3,600/RL-1RL/BX	
TD-MID-3BBD		3"	3,600/RL-1RL/BX	
DPS-MIT-3BD		3"	3,600/RL-1RL/BX	Direct Thermal IR/Permanent




## Cerner V-Notch™ Labels

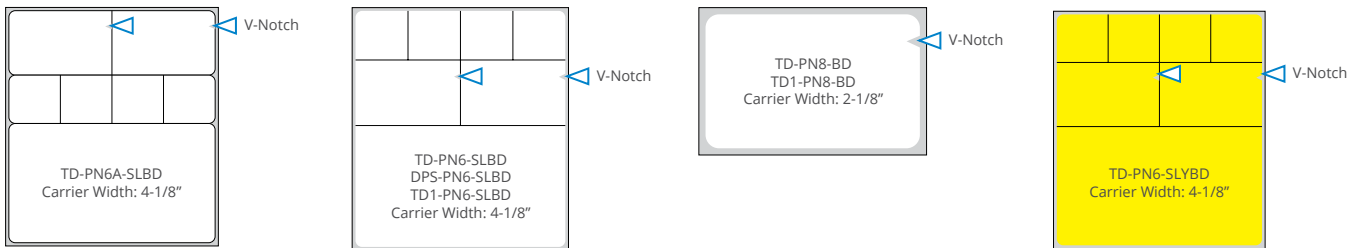


PRODUCT NUMBER	LABEL SIZE	CORE SIZE	QTY.	MATERIAL/ADHESIVE
TD-PN6A-SLBD	4" X 4-1/2"	3"	1,000/RL-2RLS/BX	Direct Thermal/Permanent
TD-PN6-SLBD	4" X 4-1/2"	3"	1,000/RL-2RLS/BX	
TD1-PN6-SLBD	4" X 4-1/2"	1"	500/RL-4RLS/BX	
TD-PN6-SLYBD	4" X 4-1/2"	3"	1,000/RL-2RLS/BX	
TD-PN8-BD	2" x 1-1/4"	3"	3,000/RL-6RLS/BX	
TD1-PN8-BD	2" x 1-1/4"	1"	1,000/RL-6RLS/BX	Direct Thermal IR/Permanent
DPS-PN6-SLBD	4" x 4-1/2"	3"	1,000/RL-2RLS/BX	

## Sunquest V-Notch™ Labels

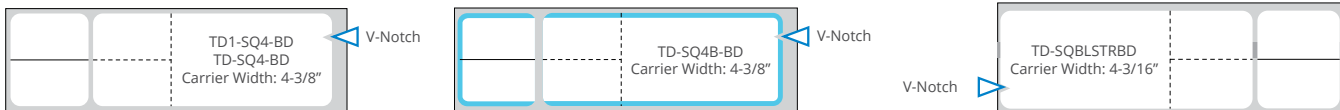


PRODUCT NUMBER	LABEL SIZE	CORE SIZE	QTY.	MATERIAL/ADHESIVE
TD1-SQ4-BD	4-1/8" x 1-3/16"	1"	1,000/RL-6RLS/BX	Direct Thermal/Permanent
TD-SQ4-BD		3"	4,300/RL-2RLS/BX	
TD-SQ4B-BD		3"	4,300/RL-2RLS/BX	
TD-SQBLSTRBD		1-1/2"	1,200/RL-8RLS/BX	

# SMS V-Notch™ Label

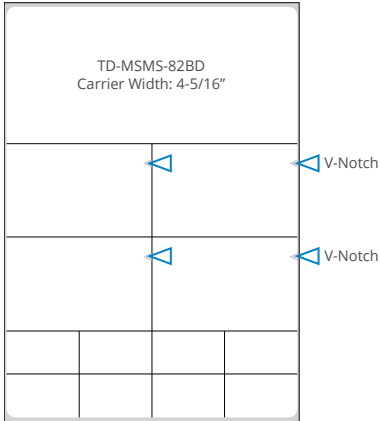

PRODUCT NUMBER	LABEL SIZE	CORE SIZE	MATERIAL/ADHESIVE	QTY.
TD-MSMS-82BD	4-1/4" x 6"	3"	Direct Thermal/Permanent	500/RL-4RLS/BX

# V-Notch™ Labels for Point of Care

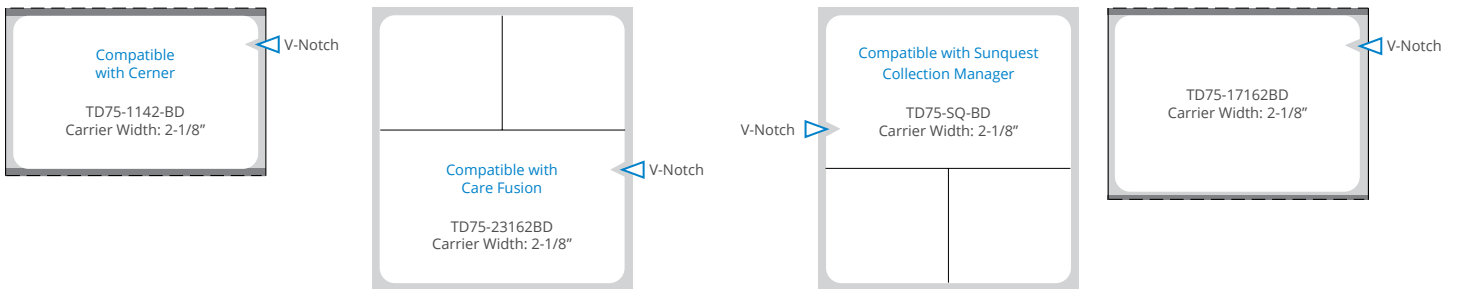

PRODUCT NUMBER	LABEL SIZE	CORE SIZE	MAX OD	MATERIAL/ADHESIVE	QTY.
TD75-1142-BD	2" x 1-1/4"	3/4"	2"	Direct Thermal/Permanent	250/RL-8RLS/BX
TD75-23162BD	2" x 2-3/16"				200/RL-8RLS/BX
TD75-SQ-BD	2" x 2-3/16"				200/RL-8RLS/BX
TD75-17162BD	2" x 1-7/16"				250/RL-8RLS/BX

**Get V-Notch™ on ANY Label Configuration!**  
 Don't see your LIS format? No problem. V-Notch is available on any label configuration. Call 800.435.4242 for more information.

Learn more.  
 Visit [pdchealthcare.com/v-notch](http://pdchealthcare.com/v-notch)

PDC 25124 Springfield Court, Suite 200, Valencia, CA 91355  
 Tel 800 435 4242 | Fax 800 321 4409 | [info@pdchealthcare.com](mailto:info@pdchealthcare.com) | [pdchealthcare.com](http://pdchealthcare.com)

